


MRI —



SPECIFICATION DATA

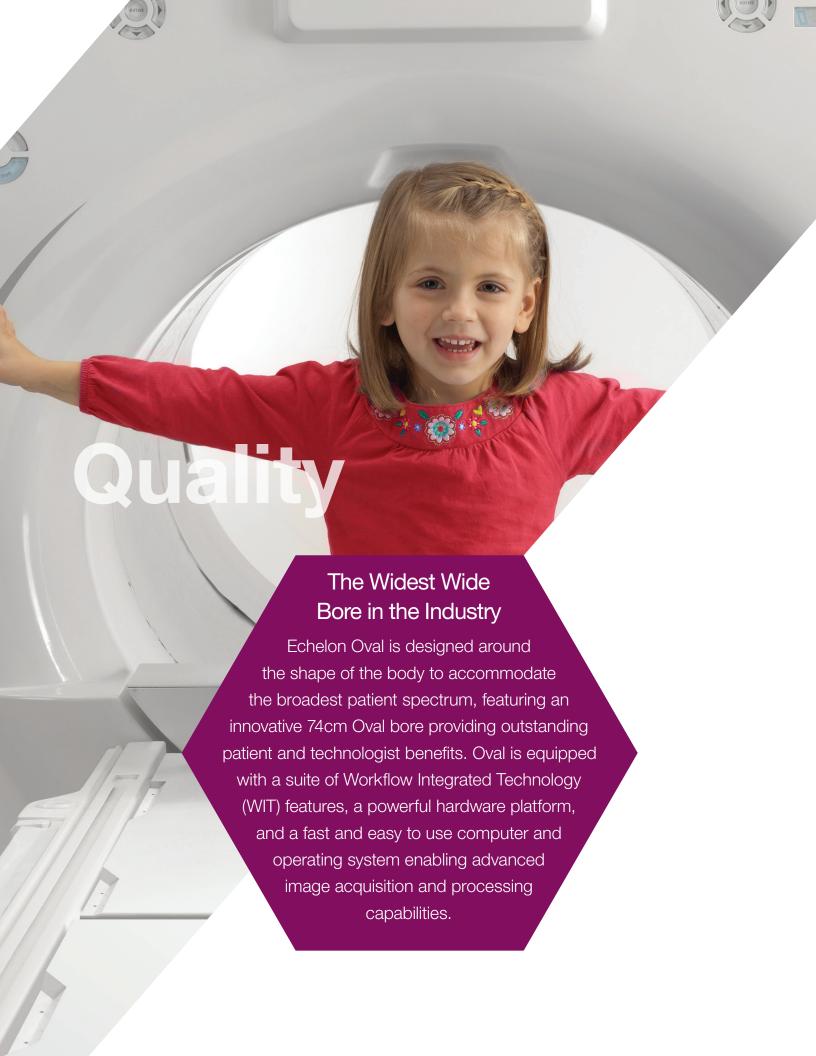







# **Key Components and Specifications**

- 1.5T Superconducting Magnet with 74cm Oval Bore
- Patient-Centric Management System
  - WIT Mobile Table
  - WIT Gantry Monitor
- Gradient System 34mT/m and 150T/m/s
- WIT RF System with Integrated RF Receiver Coils
  - 16 and 32 channel receiver
  - Digital Drive DX
- Vertex<sup>™</sup> II Computer System and Origin<sup>™</sup> MRI Operating Software
- Imaging Suites Pulse Sequence and Aquisition Techniques
- Siting Considerations and Component Dimensions

## **Inspired By Your Patients**

## Magnet System

The Echelon Oval magnet features a 74cm oval bore for maximum patient accessibility and comfort. Echelon Oval's 1.5T magnet features high homogeneity, ultimate stability, and a full 50cm FOV in all directions. Echelon Oval also includes HOAST™ (Higher Order Active Shim Technology) applied per patient assuring exceptional magnetic field uniformity. Echelon Oval features a compact footprint and virtually zero Helium boil off.

- Superconducting magnet
- 1.5 Tesla
- Horizontal field
- Homogeneity: <0.5ppm @ 40cm DSV (VRMS)</li>
- Shimming:
  - Installation: Computer mapped passive shim
  - Patient: Linear and Higher order nper patient active shim
- Active magnetic shielding
- 5G Fringe field
  - Axial: 4.0m (13.1ft)
  - Radial: 2.5m (8.2ft)
- Helium frequency: Once every six years with Fujifilm's Cusomer Support Program



## Patient Centric Management System

Fujifilm's mastery of patient-focused MRI imaging is demonstrated in Echelon Oval's attention to patient comfort. It begins with Echelon Oval's bore, the widest bore (74cm) of any 1.5T. The extra width accommodates the extra-wide WIT Mobile Table (63cm), with a 550lb weight capacity, and mobility to reduce patient transfers for easy accessibility. The table's 9+ feet of travel allows patients to enter the bore in the less stressful feet-first orientation for any exam. It lowers to 50cm allowing easy access even for wheelchair patients. Arm boards on either side accommodate patient IVs and extend vertically to ensure patient safety during transport. A positive patient experience is enhanced with adjustable airflow, lighting, and two-way communication. A gantry-mounted Workflow Integrated Technology (WIT) Monitor provides the operator with patient, coil, and gating information to further speed patient preparation.

- Patient aperture: 74cm x 65cm
- Table weight capacity: 550lb (250kg)
- Table width: 63cm
- Longitudinal travel: >9 ft (279cm)
- Vertical range: 19.7-33.2in (50-84cm)
- Class II laser positioing
  - +/-0.5 mm accuracy
  - Automatic movement to ispcenter



Comfort

- Table control
  - Up/Down
  - In/Out (Slow/Fast)
  - Table position in mm
  - Move to isocenter
  - Return to zero position
  - Stop
  - Release
  - Laser
  - Clear
  - Scan control
  - Start/Abort/Pause
- Patient amenities
  - Two-way intercom
  - Technologist alert system
  - Adjustable bore illumination
  - Adjustable bore ventilation
  - Patient pads and immobilization straps





## **Gradient System**

High gradient performance is key to high performance imaging. The Echelon Oval includes a 34/150 capable gradient system. This high slew rate enables selection of low TR, TE, and IET in combination with small FOV and thin slices. This level of gradient capability positions Echelon Oval to adapt to changing MRI technology and widening applications far into the future.

Peak amplitude: 34 mT/mPeak slew rate: 150 T/m/s

Active shieldingWater cooling

Gradient noise reduction: Mechanical gradient sound dampening

## Radiofrequency System with WIT Receiver Coils

Echelon Oval's RF receiver system manages multiple coil connection points on the table. The WIT RF coil system provides integrated coil arrays that can be used individually or in combination to give the operator maximum flexibility for positioning patients of all sizes. The WIT Spine coils are plugged directly into the table, so no cables get in the way. The WIT Posterior Head/Neck array can be used at either end of the table permitting head-first or feet-first imaging for all exams. The WIT receive coil system includes anterior Head, Neck, Neurovascular, Torso, and Cardiac arrays that can be easily placed on the patient for imaging.

Virtually all of Echelon Oval's array, surface, and volumetric coils are multiple element designs for high signal uniformity, high SNR, and compatibility with RAPID<sup>TM</sup> parallel imaging for maximum clinical flexibility and image quality. Analog to Digital conversion in the scan room with optical digital transmission of MRI signal data prevents electrical noise pickup and ensures highest possible SNR.

#### **RF Transmit:**

Two 20kWRF power amplifiers

### Digital Drive Dx Receiver:

- 16 or 32 channels
- 7 coil connection points
- Ultra low noise coil mounted preamplifiers
- A/D conversion on gantry with optical digital transmission to equipment room

#### Available Coil Set Includes:

- WIT Posterior Head/Neck can be used at either end of the table for maximum patient comfort and technologist convenience
- WIT Posterior Spine A and B coils plug directly (no cable) into the patient table to maximize contact with the patient
- WIT Anterior Head
- WIT Anterior Neck
- WIT Anterior Neurovascular
- WIT Torso
- WIT Cardiac



- Shoulder
- Extremity
- Hand/Wrist
- Foot/Ankle
- Felxible Extremity
- Bilateral Breast
- Peripheral Vascualr
- Extremity Micro Coil
- QD T/R Body

## Workflow & Efficiency

## Vertex II Computer System with Origin MRI Operating Software

From patient registration through scan planning, scanning, image processing, and image management, Echelon Oval's Vertex II computer and Origin MRI operating software deliver seamless workflow. The Origin Clinical Study Library, Graphical User Interface (GUI), Intelligent Parameter Guidance, and Real-time Image Quality Calculator make scan planning a breeze for even the most complex examinations. Simultaneous scan, reconstruction, and multi-tasked image processing keep patient volume high, and Workflow Plus™ interoperability features ensure seamless Echelon Oval-HIS/RIS integration. With Echelon Oval, your operational efficiency is assured.

#### **VERTEX II Workstation**

Fast GUI, simultaneous scan and reconstruction drive high workflow efficiency.

#### CPU:

- Core i5 Processor
- 8 GB RAM
- Display
  - 24in LCD color monitor
  - Display matrix 1920x1200
- Magnetic disk:
  - 320GB storage capacity
  - Stores up to 400,000 images (256x256)

#### DVD archive

- Media capacity: 4.7 GB
- Stores up to 30,000 images (256x256)
- CD/DVD writer with autolaunching PC viewer software\*

## Scan/Reconstruction Engine:


- Multiple processors
  - Pulse sequence control
  - Digital receive
  - Image reconstruction
  - Post- image reconstruction
  - Simultaneous scan and reconstruction

## **ORIGIN MRI Operating Software**

- Log-on security features
  - Login with password
  - Normal and Audit user privileges
  - Timeout
  - Audit log
- Patient information management
  - Registration window
    - User-defined data fields
    - Automated study ID assignment
    - Rapid registration mode
  - Registration from HIS/RIS
  - Patient data correction feature
- Exam window
  - Multiple viewports for easy setup
  - 2-point and 3-point positioning
  - Multi-angle positioning
  - Image centering function
  - Interactive scan
  - Easy sequence selection and parameter adjustment
  - Basic and advanced parameter screens
  - Preview window for quick review of completed scans
  - Independent patient windows

- Patient directory
  - Directory management through drag and drop
  - Patient/study view
  - Modality Worklist Management
  - Search capability
- MRI software launcher

- Protocol library organized by anatomical groups
  - Fujifilm provided recommended protocols
  - User-defined custom protocols
- Graphical selection

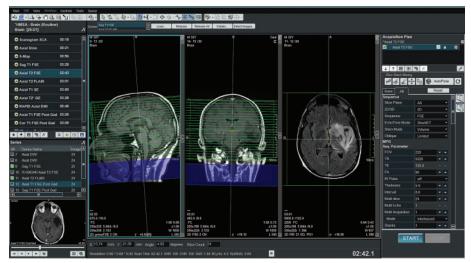

The powerful Vertex II Workstation with Origin MRI operating software easily manages multiple patients and tasks simultaneously.



- Processing tasks
  - Max/Min Intensity Projection (MIP/minIP)
  - Multi-Planar Reconstruction (MPR)
  - Vascular Volume Rendering
  - Signal Intensity Ratio Map (SIR Map)
  - Addition/subtraction
  - T1 and T2 calculated Images
  - T2 RelaxMap
  - T2\* RelaxMap
  - Dynamic analysis
  - Perfusion analysis
  - Diffusion analysis
    - Single direction analysis
    - Multi direction analysis
      - ADC trace
      - DWI trace
  - Tensor/Kurtosis analysis
    - Mean Diffusivity (MD)
    - Fractional Anisotropy (FA)
    - DWI trace
  - Post-Reconstruction functions
    - Filtering
  - Spectroscopy analysis
    - Single voxel
    - Dual voxel
    - Multi-voxel (CSI)
    - Breast Spectroscopy

- Film, Archive, and Network Functions
  - Flexible filming options
  - Drag-and-Drop Archiving/Restoring
  - DICOM 3.0 Compliant
    - Print
    - Query/Retrieve
    - Storage
    - Storage Commitment
    - Modality Worklist Management
    - Modality Performed Procedure Step
  - IHE Profiles
    - SWF/PIR
    - CPI
    - KIN
    - Basic Security
- Image review tools
  - WW/WL
    - Magnify
    - Pan
- Comment/Label

- Cine

- Measurement

- ROI Statistics
- Image Rotation
- Sentinel™ Remote Customer Support
  - Remote system and cryogen monitoring
  - Remote desktop
  - Remote diagnostics
  - Remote image review



## **Clinical Capabilities**

## **Imaging Suites**

The powerful Echelon Oval imaging architecture delivers outstanding clinical imaging benefits through the Imaging Suites. The standard Imaging Suites include a broad range of acquisition sequences, sequence enhancements, and processing tools. Scanning and processing features are available to meet the clinical challenge in Neuro, Orthopedic, Body, Breast, Vascular, and Cardiac imaging.

### Pulse Sequences

General to advanced, the acquisition sequences you need to meet your clinical challenge.

- Spin Echo (SE)
  - Up to 4 echoes
- Inversion Recovery (IR)
  - FLAIR
  - STIR
  - Magnitude and Real (Real-IR) reconstruction
- 2D/3D Fast Spin Echo (FSE)
  - Echo Factors (ETL): 2-256
  - User defined inter-echo time
  - User defined echo allocation
    - Centric
    - Anti-centric
    - ADA
    - Sequential
  - Single Shot FSE—ultra fast high echo factor acquisition for MRCP, Urography, and Myelography
  - Driven Equilibrium—Increases SNR and contrast over conventional FSE without increasing TR
- opFSE—optimized image clarity, contrast and SNR
- primeFSE—user selectable receiver bandwidth
- isoFSE—3D isotropic acquisition (T1, T2, PD, IR)
- Fast Inversion Recovery (FIR)
  - Echo Factors: 2-256
  - Inversion Time: 20-8,000
  - Driven Equilibrium
  - primeFIR
  - Double and Triple IR Black Blood acquisitions
- 2D/3D Gradient Echo (GE) and Multi-Echo Gradient Echo
- Micro TE—< 1ms TE acquisition</li>
- ADAGE—combined echo imaging for high T2\* contrast
- 3D GEIR—combined with an IR pulse for an isotropic acquisition
- FatSep Fat Separation (Dixon)
  - 2-point RSSG
  - 2 or 3-point FSE

- RADAR Motion Compensation
  - Spin Echo BASG
  - FSE GE
  - FIR/FLAIR TOF
- RAPID Parallel Imaging Acceleration
  - Image Based
  - K-space Based
- TIGRE™—3D volume gradient echo with RF fat saturation
- 2D/3D Steady-State Acquisition Rewound Gradient Echo (SARGE SG)
  - RF-Spoiled SG (RSSG)—provides T1 weighted imaging
  - Rephased SG-flow compensation for reduced artifacts
  - Balanced SG (BASG)—provides high SNR and bright fluids
  - Phase Balanced SG (PBSG)
  - Phase-cycled fat suppression cardiac imaging
  - Time Reversed SG (TRSG)—T2 weighted fluoro imaging
- Diffusion Weighted Imaging (DWI)
  - Single Shot SE EPI
    - Multi B-Factor: 0-2,000
    - RF fat saturation
    - IR pulse
- Diffusion Tensor/Kurtosis—up to 30 axes
- Perfusion
  - Dynamic Susceptibility Contrast (DSC)
  - ASL Perfusion (non-contrast)
- BSI (3D multi-shot gradient echo EPI)
  - Contrast from tissue susceptibility differences
- 2D/3D TOF
- BeamSat TOF and VASC-ASL—selective cylindrical beam saturation
- FLUTE™—fluoro triggered MRA
- TRAQ™—time resolved MRA
- Phase Contrast MRA (PC-MRA)
  - Velocity encode: 5-400cm/sec, increment 1cm/sec
- Non-Contrast MRA
  - VASC™-BASG with walking pre-sat
  - VASC-ASL-arterial spin labeling method
  - VASC-FSE—gated acquisition with image subtraction





## Acquisition Features and Protocol Enhancements


Scan fast and deliver excellent results using these pulse sequence enhancements and features designed to minimize artifacts and increase ease-of-use.


- Image plane selection
  - Transverse, Sagittal, and Coronal
  - Single and Double Oblique
  - Multi-slice, Multi-angle
  - Radial for simplified MRCP, Knee acquisition planning
  - Multi-plane for combined Sagittal, Coronal, Axial acquisition (SC, SCA, CA, or SA)
  - Interactive Scan Control (I-Scan) enables efficient plane selection and real-time image collection with slice position, scan parameter change and update for MRI Fluoro
  - AutoPose: Automatic slice planning for brain imaging
- Prescan
  - RF power adjustment
  - Center frequency
  - Volume shim adjust
- User defined regional shim
- Image Processing Algorithm
  - Adjustable image quality parameter
  - Increases image clarity and sharpness
- NATURAL™ image quality enhancement algorithm
- Coil mode search optimizes SNR for multiple coil usage
- Real-time image quality indicator (relative SNR, CNR)
- Real-time spatial resolution update shows impact of parameter changes prior to scanning
- Image centering—Places center of prescribed slab at magnet isocenter automatically for best image quality
- Auto voice
- Dynamic scan time table provides graphical review of dynamic scan procedure (steps and timing) for simplified study planning
- Imaging ParametersSlice thicknessSignals averaged: 1–99
  - 3D multi-slab: 32
  - Maximum number of 2D slices
    - 256 (512x512)
  - Maximum number of 3D slices
    - 512 (512 × 512)
    - Acquisition matrices
    - Up to 1024 × 1024
  - Reconstruction matrices
    - Up to 2048 × 2048
    - Flexible Recon Matrix

- Motion compensation Fat suppression techniques
  - RADAR radial acquisition (FSE, FIR, FLAIR, DWI, SE, primeFSE, BASG, GE, TOF)
  - Gradient rephasing
  - Presaturation pulses-up to eight
  - Walking presaturation
  - Cardiac gating with arrhythmia rejection
  - Cardiac retrospective gating
  - Peripheral Pulse Gating with arrhythmia rejection
  - Respiratory gating
  - Diaphragm Navigation Echo
  - Intermittent presaturation
  - Beam Navi
- Fat suppression techniques
  - SINC RF fat saturation (conventional SINC pulse)
  - H-SINC RF fat saturation (Light mode for lipid only, Heavy mode for lipid and olefinic suppression)
  - FatSep
  - Water Excitation (Binomial technique)
  - STIR, Fast STIR (FIR)
  - In/out of phase GE
- User defined variable bandwidth
- Dual Slice acquisition
- Rectangular Field of View
- Anti-aliasing
- User defined inter-echo spacing
- Half Scan and 3/4 scan
- Half Echo

BSI

- Asymmetric Measurement Imaging (AMI)
- Quantitative Mapping
  - T2 RelaxMap—cartilage map
  - T2\* RelaxMap—liver iron map
  - SIR Map-carotid plaque

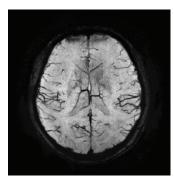

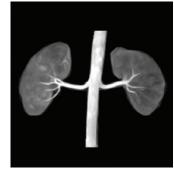

ASC ASL

- Slice thickness2D: 0.5–100mm
  - 3D: 0.05-10mm
- FOV: 3-50cm
- TR: 0.9–20,000msTE: 0.25–7,680ms
- TI: 20–8.000ms
- Inter-echo time (IET)
  - FSE: 4.4-30ms
  - EPI: 0.4-7ms
- Flip angle (FA)SE: 3–120
  - GE: 3-90

## **Oval Low Cost of Ownership**

Echelon Oval continues the Fujifilm tradition of advancing MRI systems beyond the technology you expect with cost-effective siting and operation. Echelon Oval's remarkable design attributes make it accommodating to existing facilities and easily planned into new construction. As an acknowledged leader in imaging placements, Fujifilm offers a wealth of site planning experience and a proven system for efficient siting, installation, and start-up.

## Siting Considerations:

- Typical room size
  - Scan room
    - 23.8' × 15.0' (7.2m × 4.6m)
    - Min. ceiling height: 8.7' (2.65m)
  - Equipment room
    - 13.2' × 9.0' (4.0m × 2.7m)
    - Min. ceiling height: 8.2' (2.5m)
  - Control room
    - 10.1' × 9.0' (3.1m × 2.7m)
    - Min. ceiling height: 8.2' (2.5m)
  - 5 gauss line magnetic leakage flux
    - Axial: 13.1' (4.0m)
    - Radial: 8.2' (2.5m)

- RF-shielding scan room
  - RF noise <0dB μV/m from 10-80 MHz
- AC power
  - Voltage: 3 phase AC 460V, 480V (60Hz)
  - Frequency 50/60 Hz +/-1% or less
  - Capacity 100 kVA
- Air conditioning
  - Scan room
    - Ambient operating temp: 68–75°F (20–24°C)
  - Equipment room
    - Ambient operating temp: 64–75°F (18–24°C)
  - Control room
    - Ambient operating temp: 64–79°F (18–26°C)





## **Component Dimensions**

Gantry

Length: 70.5in (179cm)Width: 86.6in (220cm)Height: 86.6in (220cm)

- Weight: 9,680lbs (4,400kg)

Bore

Oval design: 74cm × 65cmLength: <63in (160cm)</li>

Patient table

Length: 89in (226cm)Width: 29.5in (75cm)

- Tabletop Width: 25in (63cm

- Height

Max: 33.2in (84cm)Min: 20in (50cm)

Computer

- QWERTY keyboard

- 2-button mouse with scroll

LCD monitor

- 24in LCD monitor

Switch/Microphone

- Scan control

- Patient intercom

RFIP cabinet

Width: 32.3in (82cm)Depth: 39.4in (100cm)Height: 74in (188cm)

GCPA cabinet

Width: 30.7in (78cm)Depth: 44.3in (113cm)Height: 74in (188cm)

Helium compressor

- Width: 20.8in (53cm)

- Depth: 17.4in (44cm)

- Height: 24.6in (62cm)

Heat exchanger


Width: 30in (76cm)Depth: 28in (71cm)

- Height: 34in (86cm)

Sense unit

Width: 31.4in (80cm)Depth: 15.7in (40cm)

- Height: 41.7in (106cm)

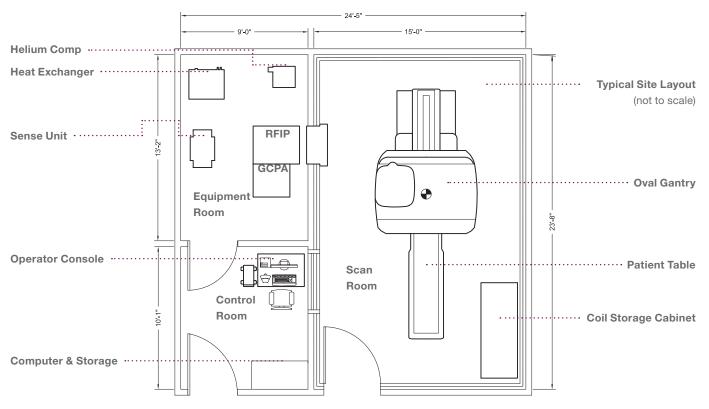

Echelon Oval Site Plan (578sq.ft.)



Fujifilm reserves the right to change specifications described herein without prior notice. This document provides general technical descriptions of both optional and standard features.

### **FUJIFILM Healthcare Americas Corporation**

81 Hartwell Avenue, Suite 300, Lexington, MA 02421 www.fujifilmusa.com 800.431.1850 ©2021 FUJIFILM Healthcare Americas Corporation

